



Sonic Alert

**REMOTE RECEIVER
MODEL SA101**

OPERATION

The Remote Receiver SA101, will alert you by flashing a light when a signal is transmitted from a Sonic Alert Transmitter (Telephone Ring Signalers, Baby Cry Signaler or Doorbell Signaler). A signal from one or all of the transmitters will be received by the SA101 and in turn will flash a light which is plugged directly into the SA101. The SA101 will have a different sequence of flashing depending of which transmitter is being activated.

INSTALLATION

1. Plug a lamp into the built-in outlet on the SA101. **Make sure the switch on the lamp is in the "ON" position or the lamp will not flash when the telephone rings.**
2. Plug the SA101 Remote Receiver into any standard electrical outlet.

Sonic Alert's Limited Warranty

The Remote Receiver, Model SA101 is warranted against manufacturing defects in materials and workmanship for two (2) year from the date of purchase. Within this period Sonic Alert will repair or replace at out option the SA101 without charge or parts and labor. Simply send the SA101 (postpaid) and a copy of your sales slip as proof of purchase to: Sonic Alert Inc., 1050 E. Maple Road, Troy, MI 48083.



Sonic Alert
REMOTE RECEIVER
MODEL SA101

OPERATION

The Remote Receiver SA101, will alert you by flashing a light when a signal is transmitted from a Sonic Alert Transmitter (Telephone Ring Signalers, Baby Cry Signaler or Doorbell Signaler). A signal from one or all of the transmitters will be received by the SA101 and in turn will flash a light which is plugged directly into the SA101. The SA101 will have a different sequence of flashing depending of which transmitter is being activated

INSTALLATION

1. Plug a lamp into the built-in on the SA101. **Make sure the switch on the lamp is in the "ON" position or the lamp will not flash when the telephone rings.**
2. Plug the SA101 Remote Receiver into any standard electrical outlet.

Sonic Alert's Limited Warranty

The Remote Receiver, Model SA101 is warranted against manufacturing defects in materials and workmanship for two (2) year from the date of purchase. Within this period Sonic Alert will repair or replace at out option the SA101 without charge or parts and labor. Simply send the SA101 (postpaid) and a copy of your sales slip as proof of purchase to: Sonic Alert Inc., 1050 E. Maple Road, Troy, MI 48083.

